

CROWNS FOR FUTURE PARTIAL (ios or pvs)

1. Send a full arch pvs impression, bite registration and opposing cast. Please make sure that all the teeth are present on the arch so it can be surveyed for proper clasp design.
***take a triple tray back up impression if you don't like the margins

OR/PRIOR TO PREP

2. Send study casts for future planning which include the design and cost estimate.

CROWNS TO FIT EXISTING PARTIAL (pvs only)

Reline the partial first if it is not fitting correctly.

***Needs to be completed prior to the prep has begun

1. Take a pvs impression with the partial in place, DO NOT REMOVE THE PARTIAL FROM THE IMPRESSION. Pour up the impression or send it to our lab as is and we will pour it up. **remember. . . patient will be without their partial during fabrication time. Contact the lab for turnaround time frame.*
***take a triple tray back up impression if you don't like the margins
2. Take a full arch impression of the opposer.
3. Take two bite registrations: one with the partial in place and one without it in place.



GNATHODONTICS, LTD